

Sami[®]


WEARABLE
User Manual
WS-2334

TRACKER

★ SMART BAND ★

Thank you for purchasing the **Sami** wearable WS-2334. Read the instructions carefully to obtain optimum results and preserve it for future reference. **Sami** has developed wearable devices since 2015 and we use the best components possible for the product.

DESCRIPTION OF THE WEARABLE

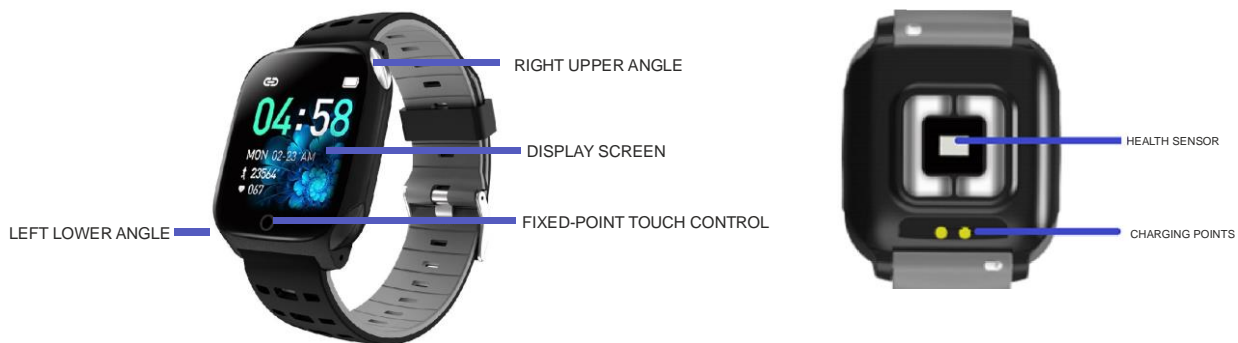

CONTENTS

Wearable, USB Magnetic cable, instruction manual.

CHARGING

Connect the device to a computer or laptop with the USB Magnetic cable for charging. Put in contact the magnetic end of the usb cable with the charging points positioned in the back of your device. Kindly use only the USB cable included with the wearable. If you use a power adapter for charging, the output of the power adapter should be 5V, 1A. Charging time: 2 hours. Please charge the device fully before using it. Take care of the magnet side of the cable, if connect to AC and you have contact with a metal surface, you may damage the charging cable.



HOW TO POWER ON / POWER OFF?

- 1) To turn ON the smart band, on the screen, long press the touch button (fixed-point touch control) for a few seconds. The touch button is situated on the bottom of the screen.
- 2) To turn OFF the smart band, press the touch button until you get in the MORE menu, then long press the touch button to enter. Tap again the touch button until the Power Off icon displays in the screen (🔌), long press the touch button for a few seconds and the device will turn off.

HOW TO CONTROL?

On the display screen, short press the touch button to change between the different menus. Long press the touch button to achieve the different functions.

GETTING STARTED:

- For Android

1) How to obtain the BT Connection APK (**JYOUPRO**)

Search for the **JYOUPRO APP** in the Play Store; download it and install it in your Android smartphone, or go to our **website www.sami.es** and download the application.

2) Installation of the **JYOUPRO** APK.

When installing the **JYOUPRO** application on your Android, you must allow all the Application Permissions, allow this **JYOUPRO** app auto run. The first time you run this **JYOUPRO**, it will ask you to Enable Notifications, please tap OK and tap the **JYOUPRO** and confirm OK. You must tap OK/Allow for any pop-up request. On your Android permission management, allow all application permissions.

3) How to make the wearable pair with your Android smart phone?

*Firstly, turn on the Bluetooth of your Phone.

*Secondly, run the **JYOUPRO** app on your Phone, on the home page of **JYOUPRO**, select "**Scanning device**", the APP will search for new Bluetooth device(s), when the available Bluetooth device displays, please select **F16 or WS-2334** (the BT name of this device) to make it pair. The pairing will be finished in a few seconds. When you pair for the first time, the app JYOUPRO will ask you to fill some personal information such as gender/age/height/weight. When you finish select OK/Confirm/Allow for any pop-up request.

Important: To enable all notifications of different softwares such as WhatsApp, Facebook, Skype, Twitter, you will need to turn on the notifications on the APP manually. On the home page of JYOUPRO, select the icon "Device" (situated on the bottom of the screen), and turn on all the APP notifications you want to active.

- For iPhone

1) How to obtain the BT Connection APP? (**JYOUPRO**)

On iPhone, please go to APP Store and search **JYOUPRO** and install the app.

2) How to make the wearable pair with your iPhone?

*Firstly, turn on the Bluetooth of iPhone

* Secondly, run the **JYOUPRO** app on your Phone, on the home page of **JYOUPRO**, tap "**Scanning device**", the APP will search for new Bluetooth device(s), when the available Bluetooth device displays, please tap **F16 or WS-2334** (the BT name of this device) to make it pair. The pairing will be finished in a few seconds. When you pair for the first time, the app JYOUPRO will ask you to fill some personal information such as gender/age/height/weight. When you finish select OK/Confirm/Allow for any pop-up request.

Important: To enable all notifications of different softwares such as WhatsApp, Facebook, Skype, Twitter, you will need to turn on the notifications on the APP manually. On the home page of JYOUPRO, select the icon "Device" (situated on the bottom of the screen), and turn on all the APP notifications you want to active.

PRODUCT FEATURES.

BLUETOOTH (V 4.0)

TOUCH BUTTON (Fixed-point touch control). This model is not FULL touch control.

TOUCH SCREEN. touch screen 1.3"

BUILT IN VIBRATING MOTOR

BUILD IN HEART RATE SENSOR

BUILD IN GRAVITY SENSOR

GESTURE CONTROL TO LIGHT UP THE DISPLAY SCREEN

DISPLAY SCREEN INTERFACE

1) Time/Date/Battery/Steps/Heart rate displaying

2) Heart rate

3) ECG+PPG Test

4) Sleep monitor

5) Sports (different sport mode) – Walking/ Running / Bicycling / Climbing

6) More functions – Steps / Calories / Distance / Blood pressure / Blood oxygen (SPO2) /Information(Message) Find phone / Stopwatch/ Power Off

Data can be sync from the wearable to the smart phone on the **JYOUPRO** APK/APP. Your smart phone will synchronize when the Bluetooth is paired with this device.

FUNCTIONS

PHONE CALL NOTIFICATION - vibrating reminder for incoming phone calls

MESSAGE NOTIFICATION - vibrating reminder for incoming messages, including SMS, WhatsApp, Facebook, Twitter, Skype messages, etc.

CLOCK SCREENS – (Different home digital clocks) – In the home screen of the smart band, long press the fixed-point for a few seconds to change the style of the digital clock.

SLEEP MONITOR - supported on the JYOUPRO APK/APP

ALARM CLOCK - supported on the JYOUPRO APK/APP

REMOTE SELFIE - (remote control your smartphone's camera.)

Automatic Heart Rate test – supported at JyouPro APP

STOPWATCH

POWER SAVING MODE – SUPPORTED AT JYOUPRO APP

SEDENTARY REMIDER (INACTIVITY REMINDER/REMINDE TO MOVE) - supported on the JYOUPRO APK/APP

FIND THE DEVICE(Looking for the SMART BAND) - supported on the JYOUPRO APK/APP

GESTURE CONTROL (Raise your hand to light up the screen) supported on the JYOUPRO APK/APP

GPS MAP TRACE (view it on the BT APP/APK JYOUPRO). On the home page of the JYOUPRO, tap in the Exercise icon ,then select the flag icon and select Continue to view the GPS map. On the home page of the JYOUPRO, tap in the Exercise icon, then select the icon situated in the right top to view the GPS map trace history record. The trace history can be shared in Facebook, Twitter, Instagram, etc.

ECG & PPG - Test and Report generates

1) How to test ECG + PPG?

To start directly at the smart band

Wear the smart device on your wrist → on the ECG+PPG test interface of the smart device display screen, long press touch button for few seconds to start testing; once it is start, use your other hand two fingers to touch the Left Lower Angle and to touch the Right Upper Angle respectively, at the same time. Keep still and the testing will be done in approx 45 seconds. Your left hand and your right hand should not touch each other while in testing.

FitPro APP/APK to start

Wear the smart device on your wrist → on FITPRO APP → tap Home → tap ECG → tap Enter → tap Start; once it is start, use your other hand two fingers to touch the Left Lower Angle and to touch the Right Upper Angle respectively, at the same time. Keep still and the testing will be done in approx 45 seconds. Your left hand and your right hand should not touch each other while in testing.

2) Where I can read the test result and test report of ECG/PPG?

At the Fitpro APK/APP, tap Home → tap Health → tap ECG → tap Health Report → tap and select the test history to view Health Report → roll down to find the ECG report, tap it to enter to view the report. The report can be shared to Facebook, Twitter, etc.

[If the report does not appear you must sync the data tapping on the sync icon on the top right.](#)

MULTIPLE LANGUAGES

****** THIS IS NOT A MEDICAL INSTRUMENT, SO THE VALUES ARE APPROXIMATE**

TECHNICAL SPECIFICATIONS:

Display screen: Capacitive touch screen 1.3".

Screen resolution: 240x240pixels

Internal Memory: RAM 96KB, ROM 128KB

System: Nucleus

CPU: Dialog14585

Charge I/O port : USB Magnetic charging.

Battery: Built in Li-Polymer 170 mAh

Stand by time: 7~10 DAYS

Working time: 5 DAYS

Bluetooth: v4.0

Water-resist Rating: IP68 30 meters waterproof, OK for swimming, but not for professional diving. Please use only in cold water, do not use for bath with hot water as the vapor can enter in the unit and damage it.

NET Weight: 35 g

Exterior Dimensions: device 43*37.5*13.5mm; with straps 255*37.5*13.5mm.

Declaration of Conformity

Hereby we,

Name of manufacturer: SANTOSHI SL
Address: CALLE FELIPE ASENJO N.56
Zip code & City: 28947 - FUENLABRADA (MADRID)
Country: SPAIN

declare that this DoC is issued under our sole responsibility and that this product:

Product description: Smart Band
Type designation(s): WS-2334
Trademark: SAMI WEARABLE

is in conformity with the relevant Union harmonization legislation:

DIRECTIVE EMC 2014/30/EU
DIRECTIVE LVD 2014/35/EU

Standard CE (RED): DIRECTIVE 2014/53/EU
Standard RoHS: 2011/65/EU: (RoHS) Restriction of Hazardous Substance.

with reference to the following standards applied:

EN 60950-1: 2006+A11:2009+A1:2010+A12:2011+A2:2013
EN 62479:2010
EN 301 489-1 V2.1.1 (2017-02)
EN 301 489-17 V3.1.1 (2017-02)
EN 300 328 V2.1.1 (2016-11)



Signature

Full name: VISHAL MIRPURI

Position: ADMINISTRATOR

Date: 05/ 12 / 2019